

Troughing

Restorations

Implants

Apthous Ulcers

Curretage

Crown Lengthening

Frenectomy

Gingivectomy

Soft Tissue

Gingivoplasty

Specification:

Size: 26 cm x 15cm x 10 cm
Weight: 1.6 kg
Medium: GaAlAs Laser Diode
Wavelength: 810 +- 10 nanometers
Output Power: 0.1- 5 watts at the fibre tip
Operation: Continuous wave or Pulsed at 10-20hz
Pulse Width: 50 ms
Fibreoptic: 200um Single core fibre
Aiming Beam: 650 nm red Diode laser 2 mw maximum
Sterilization: Handpiece
Input Power: AC : 100 to 240v, 1a, 47-63 hz
DC : 5.1v, 6a
Calibration: External power meter
CE marked and FDA approved laser procedures



DMDS House, 18 Dover Street, Canterbury
Kent CT1 3HD United Kingdom
Tel +44 (0)1227 780009
Fax +44 (0)1227 760649
email info@quickwhite.net
www.quickwhite.info



QuickWhite™ Lase

Today's dentistry requires different techniques and advanced equipment, which is why we have developed the QuickWhite Lase, soft tissue laser.

Lasers are no longer the preserve of a handful of specialists; the QuickWhite Lase brings a highly featured, competitively priced laser within the reach of most practitioners.

Soft tissue laser techniques have developed radically over the last few years with many routine procedures being regularly performed with lasers, with extraordinary results.



QuickWhite™ Lase

- Troughing
- Restorations
- Implant Recovery
- Aphous Ulcers
- Curretage
- Crown Lengthening
- Gingivectomy
- Frenectomy
- Gingivoplasty
- Touch screen, pre-set & user procedures
- Portable, mains or battery
- Best value soft tissue laser available today

Lase
er

Troughing



Laser troughing, when used as part of a dentist's impression technique, can greatly improve long term gingival cosmetic results. Great fitting restorations with no marginal shrinkage.

Restorations

Manage gingival tissue at safe, low power settings. Coagulation of sulcular fluids assure that composite fillings can be placed in a non contaminated field with confidence.

Implants

Secure placement of implant abutments within a clear field- no tissue overgrowth or bleeding, increasing confidence in the reduction of the risk of your implant loosening.

“QuickWhite™ Lase,
the choice of the
professional”



Apthous Ulcers



Desensitise painful aphthous ulcers; pain will typically stop within hours of treatment, enhancing the healing process.

Curretage



Non-surgical periodontal treatment allowing access to deeper calculus deposits.

Crown Lengthening



Minimum pain and bleeding, no scalpel or sutures and no post operative shrinkage. Predictable results each time.

LASE

Gingivectomy



Immediate results, no scalpel or sutures. Final impressions can be taken at the same appointment, maximum convenience, maximum accuracy.

Frenectomy



Bloodless and suture free release of the maxillary and mandibular frenums. Improved access for oral hygiene, close orthodontic diastemas and improvement of impaired speech.

Gingivoplasty



Remove redundant or hyperplastic tissue within a bloodless field with minimal - if any post operative pain. Impressions can be taken at the same visit for maximum convenience.

The results achieved in the photographic records of the procedures performed, using soft tissue lasers clearly shows the astonishing results that can be achieved by employing the latest techniques.

Dental Lasers are no longer the preserve of a handful of specialists, lasers are more commonly used across the world delivering stunning results .

When you purchase a QuickWhite Lase you will receive full training and the back up that only more than 10 years supplying hi-tech dental equipment to dentists can deliver.

Buy from the professionals...

Buy from QuickWhite.



Laser